


USER MANUAL

1. Important Notice:





Try to install SafeCheck in indoor or semi-outdoor environment, please build a rain shed If planned to work in open air.

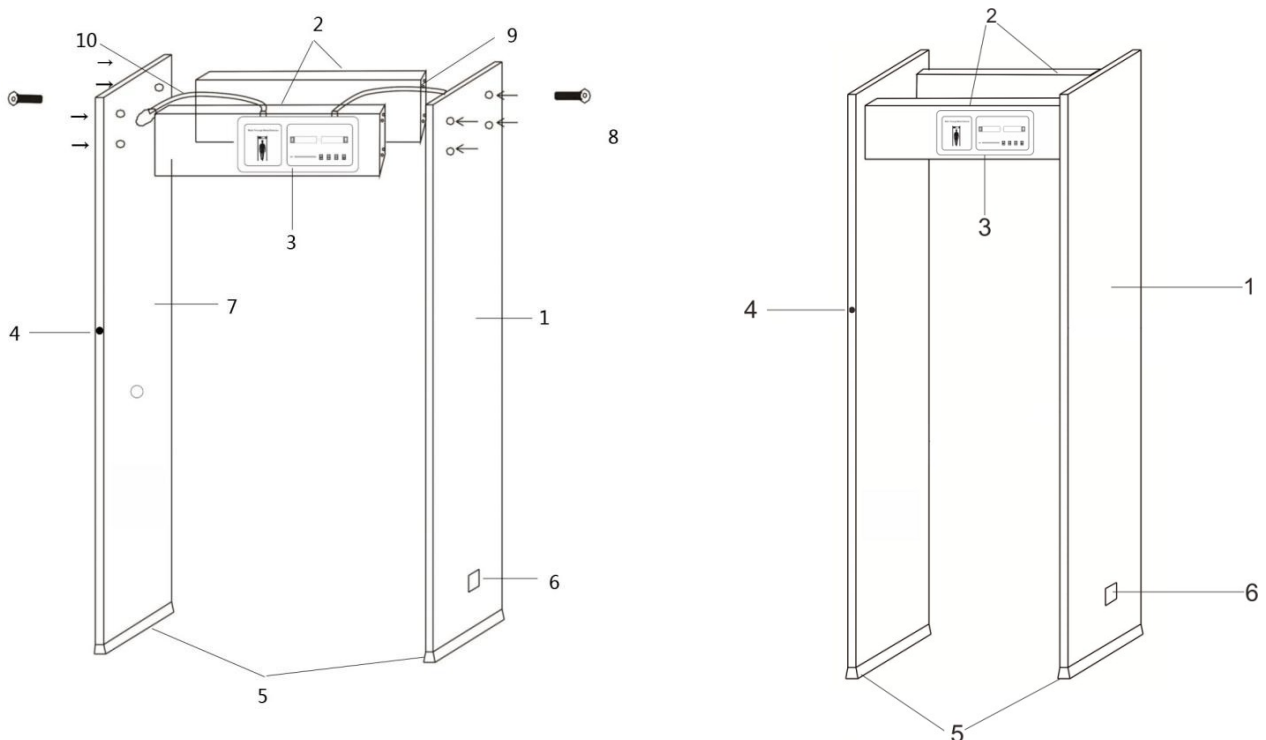
2. Installation Step

- 1) Unpack the SafeCheck.
- 2) **3-Controller** Unit facing up, place the **7-Left panel** and **1-Right panel** as shown in the figure, then connect with **8-Screw Bolt** tightly.
- 3) Plug the left and right panel **9&10-Cable** into the corresponding sockets on the panels tightly.
- 4) Raise the gate to a vertical position and move it to designated working position.
- 5) Connect the **6-Power Supply**, Turn on Power button.



1 & 7	2	3	4
Right/Left Panel	Crossbar	Control Unit	Temperature Sensor
5	6	8	9 & 10
Foot Cover	Power Supply	Screw Bolt	Connecting Cable


1. Temperature Display
2. Number of High Temperature Alarm Times.
3. High Temperature Alarm Indicator (red light)
4. “” key, to confirm the setting data
5. “” key, used as +1 when adjusting the digits
6. “” key, used as -1 when adjusting the digits.
7. “” key, **Clear** “alarm times”, but environment temperature remains.

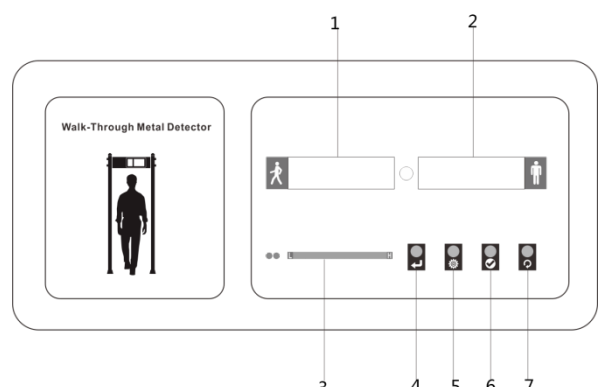

USER MANUAL





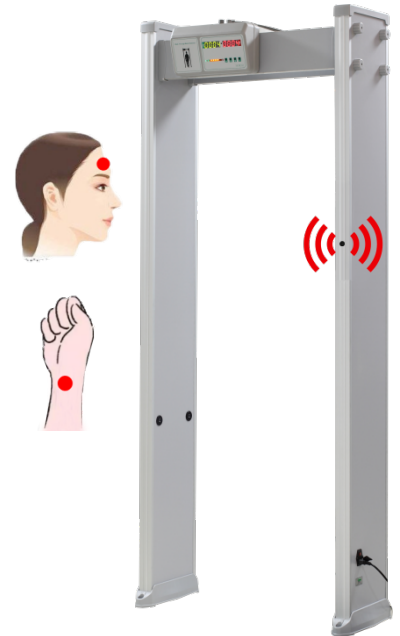



3. Temperature Detection

- 1) Present the forehead or wrist to the temperature sensor within 3 – 5 inches for accurate temperature reading.
- 2) After the inspected person passes without a sound alarm, the next person can continue to pass. If there is a sound alarm, please test again a second time for confirmation.



4. Operation Steps

Set Alarm Threshold:




1. Press “” key to show ‘1234’
2. In Order to change password from ‘1234’ to ‘5432’
(Press “” key to adjust the digit, then Press “” key to move to the second digits, then Press “” key to adjust the second digits). Repeat the process to adjust the ‘1234’ password to ‘5432’.
2. After ‘5432’ shows, Press “” key can enter the “Alarm Threshold” temperature setting.
3. Then press “”key or “” key to adjust the Alarm Threshold. Press “” key to confirm the Alarm Threshold temperature setting.




USER MANUAL


Temperature Display or Display Pass Counts:


1. Press “” key to show ‘1234’

In Order to change password from ‘1234’ to ‘9881’, Press “” key to adjust the digit, then Press “” key to move to the second digits, then Press “” key to adjust the second digits). Repeat the process to adjust the ‘1234’ password to ‘9881’

2. Press “” until the screen display a **0**, this setting will display the temperature. If the screen display shows **1** then this setting will only show the pass count and not the temperature.

Volume and Sound Adjustment

Long Press “” can adjust the volume.

Long Press “” can adjust the sound tone.